

# AbleCenter Multimedia System



## AbleGrabber™ M Set-up Instructions

- Install AbleCenter software (*AbleCenter.exe* provided on usb drive)




- Press «AbleSky» icon to create new user.




- Log in with user credentials from AbleSky
- Connect power supply to AbleGrabberM unit



- Wait until «**AbleMobile**» wireless network shows up in Wi-Fi settings
- Connect to the network with «ablecon2011» as password

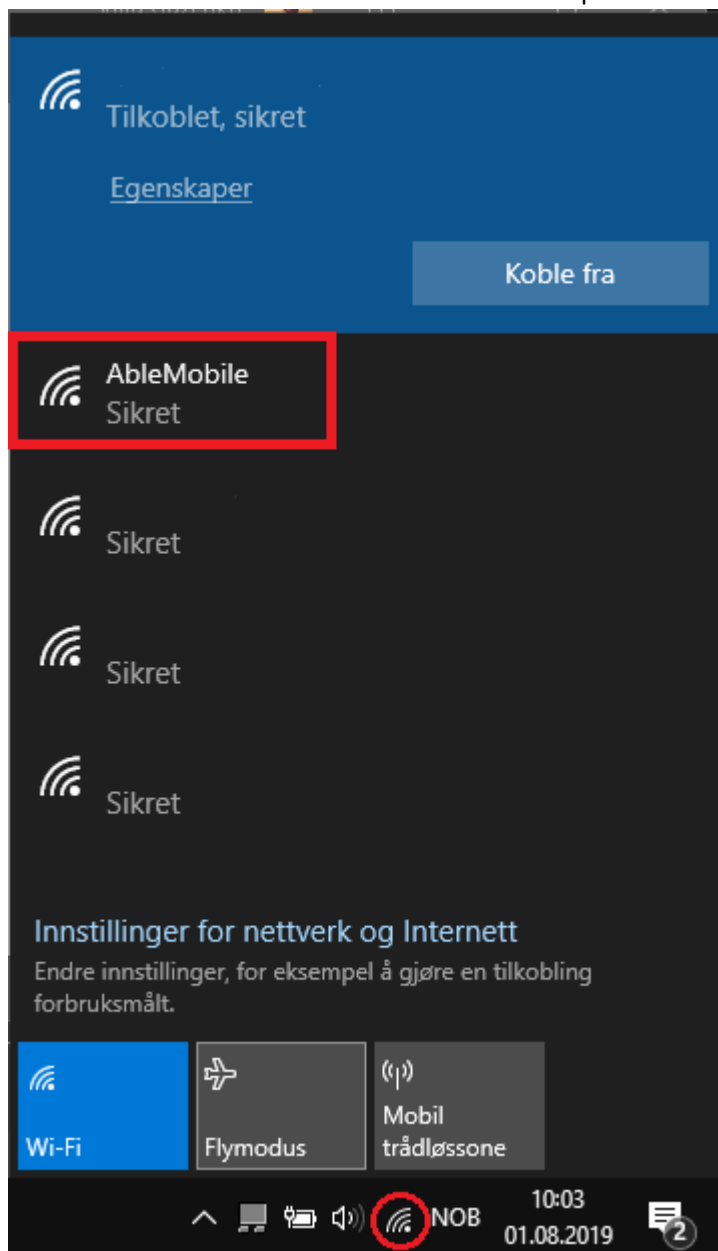

- Open AbleCenter program

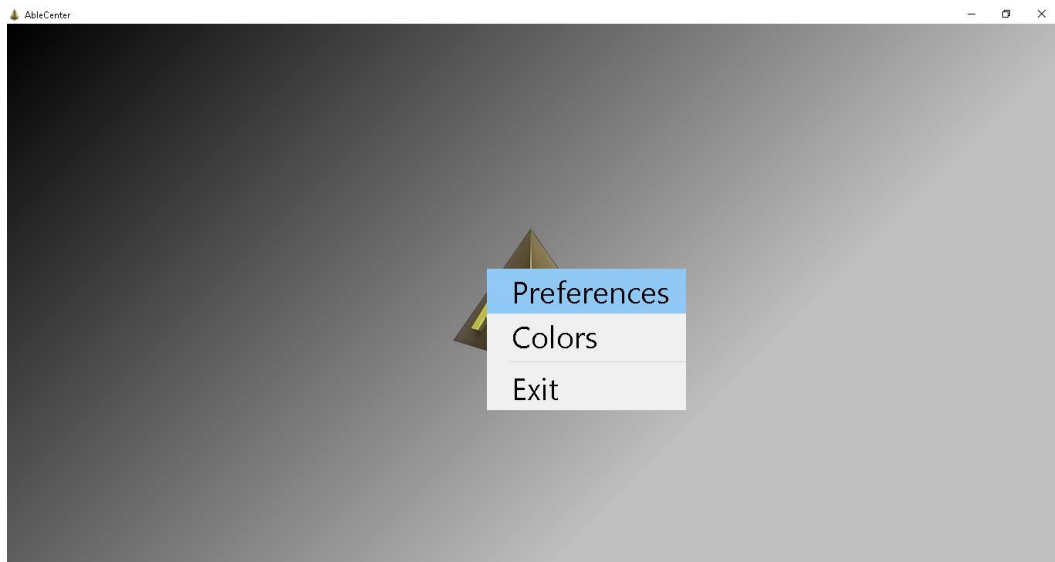

- Right click on grey screen and choose “*preferences*”
- Choose «**Advanced**» and check «**Show IP adress**»

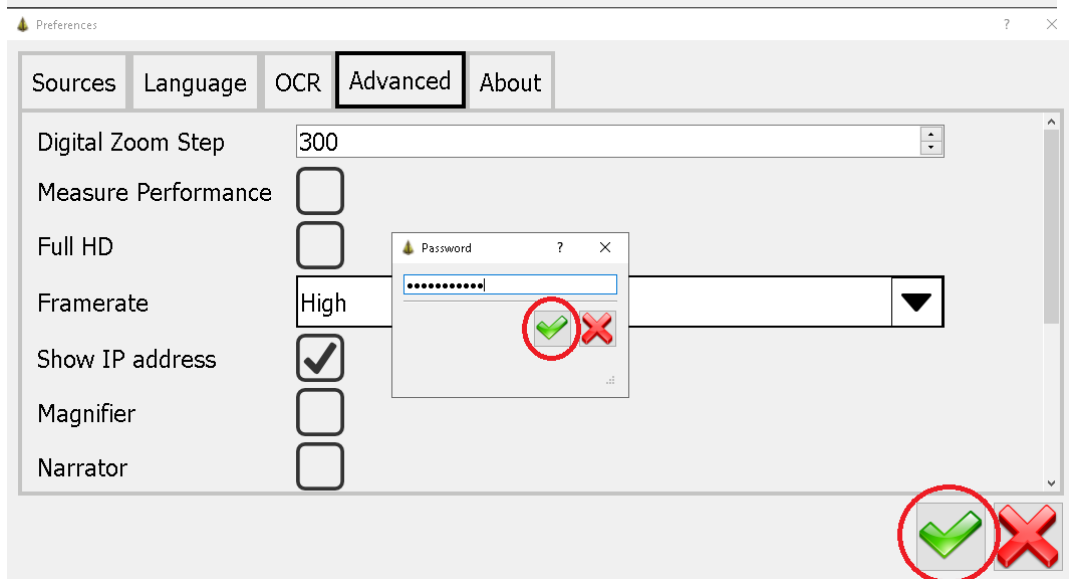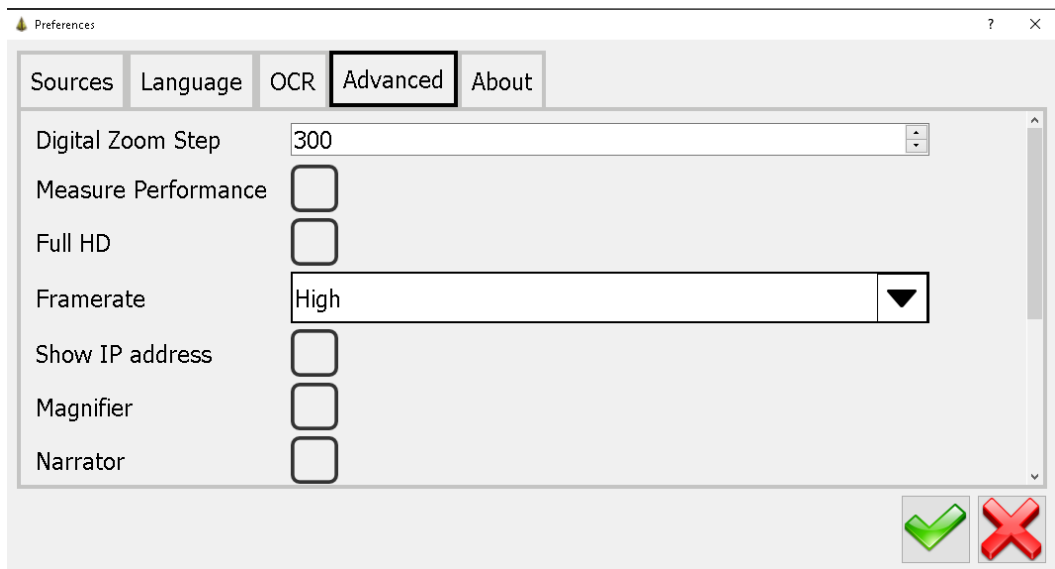

- Type in «**ablastic123**» as password

- Proceed to **“Sources”** tab
- Under **AbleGrabber** change IP address to «**192.168.250.254**»
- Make sure that only AbleGrabber is checked.

Preferences ? ×



Sources Language OCR Advanced About

☐ AbleCamera-AC2

☐ AbleCamera-AC3

☒ AbleGrabber

Reset to default IP addresses

- Confirm changes by pressing the green button in right corner.
- Restart program to get picture from AbleGrabberM